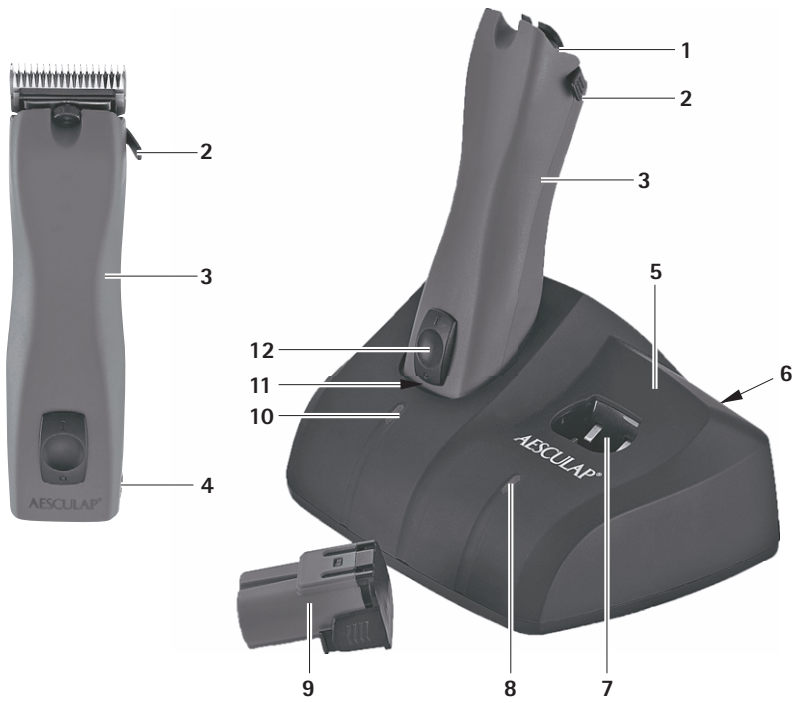
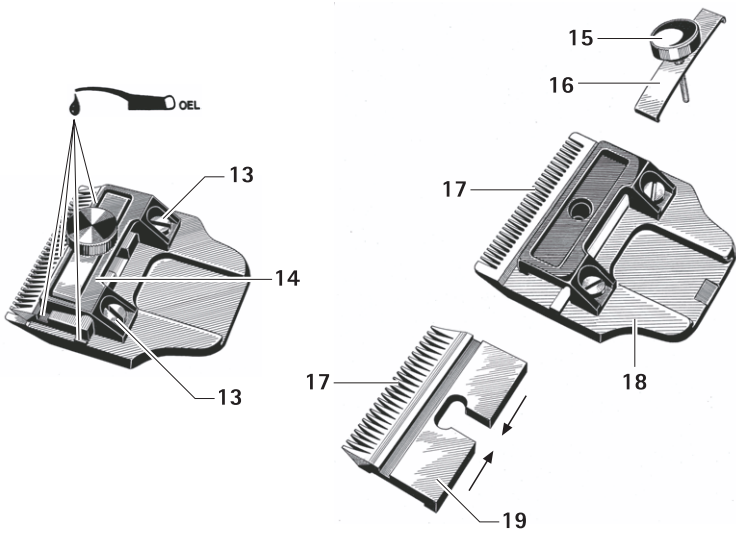
- (GB)** Instructions for use/Technical description
(USA) Battery-powered clipping machine Favorita_{CL}/Libra
- (D)** Gebrauchsanweisung/Technische Beschreibung
Akku-Schermaschine Favorita_{CL}/Libra
- (F)** Mode d'emploi/Description technique
Tondeuse à accu Favorita_{CL}/Libra
- (E)** Instrucciones de manejo/Descripción técnica
Esquiladora a batería Favorita_{CL}/Libra
- (I)** Istruzioni per l'uso/Descrizione tecnica
Tosatrice ad accumulatore Favorita_{CL}/Libra
- (NL)** Gebruiksaanwijzing/Technische Beschrijving
Accu-scheermachine Favorita_{CL}/Libra
- (S)** Bruksanvisning/Teknisk Beskrivning
Gerätebez-Titel
- (RUS)** Инструкция по применению/
Техническое описание
Машинка для стрижки, с аккумулятором,
Favorita_{CL}/Libra
- (PL)** Instrukcja użytkowania/Opis Techniczny
Akumulatorowa maszyna do strzyżenia
Favorita_{CL}/Libra
- (CN)** 使用说明书 / 技术描述
Favorita_{CL}/Libra 蓄电池剪毛机
- (J)** 取扱説明書 / 技術仕様書
充電電池駆動ヘアカッター Favorita_{CL}/Libra
- (KR)** 사용 설명서 / 기술 설명
Favorita_{CL}/Libra 충전 면도기

Battery-powered clipping machine Favorita_{CL}/Libra

Legend

- 1 Blade adapter
- 2 Lever
- 3 Battery-powered clipping machine Favorita_{CL}/Libra
- 4 Battery release lock
- 5 Charger
- 6 Mains power cord (with device plug)
- 7 Charging bay (for Li-ion battery)
- 8 Charge state indicator (for Li-ion battery)
- 9 Li-ion battery
- 10 Charge state indicator (for battery-powered clipping machine Favorita_{CL}/Libra)
- 11 Charging bay (for battery-powered clipping machine Favorita_{CL}/Libra)
- 12 On/Off switch
- 13 Screw to fix casing
- 14 Pressure piece
- 15 Knurled screw
- 16 Leaf spring
- 17 Serration
- 18 Lower cutting plate
- 19 Upper cutting plate

Symbols on product

	<p>Adhere to instructions for use</p>
	<p>Labeling of electric and electronic devices according to directive 2012/19/EU (WEEE), see Chapter Disposal</p>

Contents

- 1. Safe handling 3
- 2. Product description 3
 - 2.1 System components 3
 - 2.2 Components necessary for use 4
 - 2.3 Intended use of battery-powered clipping machine Favorita_{CL}/Libra 4
 - 2.4 Intended use of charger GT203/Li-ion battery GT201 4
 - 2.5 Operating principle of battery-powered clipping machine Favorita_{CL}/Libra 4
 - 2.6 Operating principle of charger GT203/Li-ion battery GT201 4
 - 2.6.1 Charging principle 4
 - 2.6.2 Charging time 4
- 3. Preparation and setup 5
- 4. Working with the battery-powered clipping machine Favorita_{CL}/Libra and charger GT203/Li-ion battery GT201 5
 - 4.1 System set-up 5
 - 4.1.1 Connecting the accessories..... 5
 - 4.1.2 Attaching the Blade..... 5
 - 4.1.3 Connecting the charger/Charging the Li-ion battery..... 6
 - 4.2 Function checks of battery-powered clipping machine Favorita_{CL}/Libra 6
 - 4.3 Function checks of the charger 7
 - 4.4 Safe handling: Blade 7
 - 4.4.1 Adjusting the plate pressure..... 7
 - 4.4.2 Sharpening the cutting plate 7
 - 4.4.3 Removing the blade 7
 - 4.4.4 Lubricating the blade..... 7
 - 4.4.5 Changing the Li-ion battery 8
 - 5. Processing 8
 - 5.1 Cleaning/Disinfecting 8
 - 5.2 Control, care and inspection 9
 - 6. Maintenance 10
 - 7. Troubleshooting list 11
 - 7.1 Battery-powered clipping machine Favorita_{CL}/Libra 11
 - 7.2 Charger and Li-ion battery 11
 - 8. Technical Service 13
 - 9. Accessories/Spare parts 14

10.	Technical specifications	15
10.1	Battery-powered clipping machine	
	Favorita _{CL} /Libra	15
10.2	Charger	15
10.3	Li-ion battery	15
10.4	Declaration of conformity	15
11.	Disposal	15
11.1	For EU countries only	16
11.2	Li-ion batteries	16
12.	2 Years warranty	16
13.	Distributor in the US/Contact in Canada for product information and complaints	17

1. Safe handling

CAUTION (for US users only)

Federal law restricts this device for sale by or on order of a physician!

DANGER

Risk of death by electric shock!

- Do not open the product (except for changing a charging bay).
 - Only ever connect the product to power mains with equipment grounding conductor.
- Prior to use, check for proper condition and functioning of the product.
 - Store the charger and Li-ion batteries at room temperature.
 - Keep dry the charger and Li-ion batteries.
 - Charge the Li-ion batteries prior to first use.
 - Observe our "Notes on Electromagnetic Compatibility (EMC)".
 - To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.

- Observe all safety information and maintenance advisories.
- Ensure the manufacturer's approval before using accessories that are not mentioned in the instructions for use.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical facilities in the room in which the device is being used conform to IEC norms.
- Unplug the device by pulling the plug, but never by pulling the power cord.
- Do not operate the product in explosion-hazard areas.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

Note

The device can be used by children over the age of 8 as well as by persons with reduced physical, sensory or mental capacities or with lack of experience and knowledge, if these persons are supervised or have been instructed on how to use the device safely, and if they understand the risks associated with the use of the device.

- Do not allow children to play with the device.
- Do not allow children to carry out cleaning and maintenance of the device without supervision.

Battery-powered clipping machine Favorita_{CL}/Libra

2. Product description

2.1 System components

Designation	Art. no.
Battery-powered clipping machine Favorita _{CL}	GT200
Battery-powered clipping machine Libra	GT210
Li-ion battery	GT201
Charger	GT203
Mains power cord (with device plug)	see Chapter Accessories/ Spare parts
Instructions for use	TA012167

Note
 The battery-powered clipping machine Favorita_{CL}/Libra supplied without any blade. Instead, there is a range of blades available for various applications!

2.2 Components necessary for use

- Battery-powered clipping machine Favorita_{CL}/Libra
- Blade
- Charger
- Mains power cord (with device plug)
- Li-ion battery

2.3 Intended use of battery-powered clipping machine Favorita_{CL}/Libra

Note
 The device is intended exclusively for cutting animal hair.
 The battery-powered clipping machine Favorita_{CL}/Libra is used for clipping small animals.
 The battery-powered clipping machine Favorita_{CL}/Libra must be used with a blade appropriate for the respective application.

2.4 Intended use of charger GT203/ Li-ion battery GT201

The charger GT203 is used for charging the battery-powered clipping machine Favorita_{CL}/Libra and the Li-ion battery. There is one charging bay each for the two components.

2.5 Operating principle of battery-powered clipping machine Favorita_{CL}/Libra

The battery-powered clipping machine Favorita_{CL}/Libra is switched on and off by sliding the On/Off switch up or down, respectively.

2.6 Operating principle of charger GT203/Li-ion battery GT201

The charger GT203 is designed for the mains voltage/frequency range 100 V to 240 V / 50 Hz to 60 Hz.

To make the charger GT203 ready for operation, connect it to mains power by plugging in the mains plug.

The charger GT203 is equipped with two charging bays. The first charging bay is for charging the battery-powered clipping machine Favorita_{CL}/Libra, while in the second charging bay the Li-ion battery is charged. Each charging bay has an indicator field on the charger front panel assigned to it.

Charging starts automatically, as soon as a Li-ion battery is inserted in the charging station.

The charging time in the charging bays depends on the charge state and capacity of the Li-ion batteries.

2.6.1 Charging principle

The Li-ion batteries are charged by constant current pulses.

The charge state of the Li-ion battery is monitored during charging. Monitoring of the charging curve ensures that the battery is fully charged without overcharging.

Additionally, the battery temperature and charging time are monitored as well.

The charger monitors the battery status and indicates the charge state via two LED indicators.

LED indicators:

- green = Battery fully charged
- red = Battery completely discharged

2.6.2 Charging time

Charging is stopped as soon as the maximum charging time is reached.

The charging time is approx. 50 min.

3. Preparation and setup

Non-compliance with these rules will result in complete exclusion of liability on the part of Aesculap.

- When setting up and operating the product, always observe:
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.
 - operating advisories according to IEC-/VDE regulations.

DANGER

Fire and explosion hazard!

- **Do not operate the product in explosion-hazard areas.**

CAUTION

Malfunction of, or damage to the product caused by improper setup!

- **Do not expose the product to direct sunlight or moisture.**
- **Do not cover the ventilation slots of the product.**
- **Connect the product to the specified voltage of 100 V to 240 V/50 Hz to 60 Hz.**
- **Ensure that the mains power socket used for supplying the product is freely accessible.**

Battery-powered clipping machine Favorita_{CL}/Libra

4. Working with the battery-powered clipping machine Favorita_{CL}/Libra and charger GT203/Li-ion battery GT201

4.1 System set-up

4.1.1 Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

- Please contact your B. Braun/Aesculap Partner or Aesculap Technical Service (address: see Chapter Technical Service) with any inquiries in this respect.

4.1.2 Attaching the Blade

WARNING

Risk of injury caused by sharp cutting plates!

- Apply proper care when handling and operating the blade of battery-powered clipping machine Favorita_{CL}/Libra.

Note

The battery-powered clipping machine Favorita_{CL}/Libra ships without any blade. Instead, there is a range of blades available for various applications!

- Slide On/Off switch **12** upward to switch on battery-powered clipping machine Favorita_{CL}/Libra **3**.
 - Push lever **2** towards the shell of the product.
 - Mount the blade on blade adapter **1**.
 - Release lever **2**.
- This completes the mounting of the blade.

4.1.3 Connecting the charger/Charging the Li-ion battery

WARNING

Fire hazard due to battery poles shorted by fluids or metal components!

- Do not short the Li-ion battery.

WARNING

Risk of injury and material damage due to damaged Li-ion batteries!

- Prior to charging, check the Li-ion batteries for any damage.
- Do not use or charge damaged Li-ion batteries.

CAUTION

Loss of capacity/performance due to prolonged storage of discharged Li-ion batteries!

- Store Li-ion batteries only in fully charged condition and recharge stored batteries once every month.

CAUTION

Destruction of the product, charging bay or Li-ion battery when inserting the wrong type of battery in the charging bay!

- Insert Li-ion batteries only in the appropriate charging bay.
- Insert the Li-ion battery in the correct position.

Note

Prior to battery loading, switch off the battery-powered clipping machine Favorita_{CL}/Libra!

Note

Li-ion batteries heat up slightly during charging.

Note

Unplug the mains power cord at the mains socket when the charger is not going to be used!

- To connect charger **5**: Plug in the device plug of mains power cord **6** in charger **5**.
- Plug in the mains plug of mains power cord **6** at the mains power socket.
- Insert Li-ion battery **9** in charging bay **7** of charger **5** and charge the battery.
 - or -
- Insert battery-powered clipping machine Favorita_{CL}/Libra **3** in charging bay **11** of charger **5** and charge the machine.

Charge state indicator **8** or charge state indicator **10** lights up red.

After charging, the charge state indicator **8/10** shines green and battery-powered clipping machine Favorita_{CL}/Libra **3**/Li-ion battery **9** is ready for operation.

4.2 Function checks of battery-powered clipping machine Favorita_{CL}/Libra

- Carry out a visual inspection.
- Check the switch position.
- Install the battery, see Chapter Changing the Li-ion battery.
- Mount the blade correctly and lubricate it if necessary, see Chapter Safe handling: Blade.
- Keep a charged Li-ion battery on hand, as reserve.
- Clean the battery-powered clipping machine Favorita_{CL}/Libra prior to use, see Chapter Cleaning/Disinfecting.

4.3 Function checks of the charger

- Carry out a visual inspection.

Prior to connecting the product to mains power:

- Check mains power cord **6** for possible damage.
- Check the product for any damage (e.g. bent contacts of the charging bays).
- Plug in mains power cord **6** at the device socket of the charging station.
- Plug in the mains plug at the mains power socket.
- Insert Li-ion battery **9** in charging bay **7** of charger **5** and charge the battery.
 - or -
- Insert the battery-powered clipping machine Favorita_{CL}/Libra **3** in charging bay **11** of charger **5** and charge the machine.

Charge state indicator **8** or charge state indicator **10** lights up red.

After charging, the charge state indicator **8/10** shines green and the battery-powered clipping machine Favorita_{CL}/Libra **3**/Li-ion battery **9** is ready for operation.

The product is ready for operation.

Battery-powered clipping machine Favorita_{CL}/Libra

4.4 Safe handling: Blade

4.4.1 Adjusting the plate pressure

Note

The plate pressure can be adjusted to increase the cutting power as required (see foldout page)!

- Tighten knurled screw 15 so that upper cutting plate 19 can be moved by hand, by applying moderate, lateral pressure.

4.4.2 Sharpening the cutting plate

Note

For sharpening the cutting plate, please contact Technical Service, see Chapter Technical Service.

4.4.3 Removing the blade

WARNING

Risk of injury caused by sharp cutting plates!

- Apply proper care when handling the blade of battery-powered clipping machine Favorita_{CL}/Libra.

- Loosen knurled screw 15 with leaf spring 16.
- Push upper cutting plate 19 under pressure piece 14 out of the blade.
This completes the disassembly of the blade.

4.4.4 Lubricating the blade

- Disassemble the blade, see Chapter Removing the blade.
- Clean upper cutting plate 19 and lower cutting plate 18 with a soft wipe or brush. Make certain the gaps in serration 17 are cleaned properly.
- Slightly lubricate upper cutting plate 19 and lower cutting plate 18 with special lubricant for blades GT604.

4.4.5 Changing the Li-ion battery

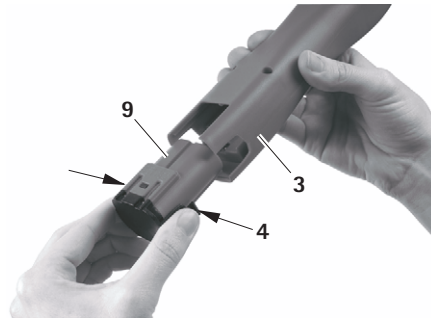
Fig. 1

- Hold battery-powered clipping machine Favorita_{CL}/Libra 3 and press battery release lock 4.
- Pull out Li-ion battery 9 from battery-powered clipping machine Favorita_{CL}/Libra 3.
- Insert the charged Li-ion battery 9 in battery-powered clipping machine Favorita_{CL}/Libra 3 as far as it will go.
The battery-powered clipping machine Favorita_{CL}/Libra 3 is ready for operation.

5. Processing

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfecting or sterilization!

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

5.1 Cleaning/Disinfecting

Note

Always follow disposal and hygiene instructions and regulations!

Manual cleaning/disinfecting

DANGER

Risk of electric shock and fire hazard!

- Unplug the unit before cleaning.
- Make certain that no fluids will penetrate the product.
- Leave the product exposed to air for at least 1 minute after cleaning/disinfecting.

CAUTION

Damage to the product due to incorrect cleaning or inappropriate cleaning/disinfecting agents!

- Use any commercial detergent for surface cleaning.
- Never immerse the product in water or detergent.
- Do not introduce any fluid into the charging bays.
- Clean the contacts in the charging bays, applying extreme care.

- Wipe the shell of the product with a lint-free cloth moistened with a commercial disinfectant for wipe disinfection.
- If necessary, after the appropriate exposure time, wipe off any residues of the cleaning/disinfecting agent with a lint-free cloth moistened with clean water.
- Dry the product with a clean, lint-free cloth.
- Clean the contacts in the charging bay with isopropanol or ethyl alcohol and a cotton swab. Do not use corrosive chemicals.
- If necessary, repeat the cleaning/disinfecting process.

Battery-powered clipping machine Favorita_{CL}/Libra

5.2 Control, care and inspection

- Allow the product to cool down to room temperature.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- Lubricate the blade as appropriate, see Chapter Lubricating the blade.
- To enhance the cutting power, adjust the plate pressure as required, see Chapter Adjusting the plate pressure.
- Check the product for any damage, a typical running noise, overheating or excessive vibration.
- Inspect the blade for broken, damaged or blunt cutting edges.
- Set aside the product if it is damaged.

6. Maintenance

To ensure reliable operation, Aesculap recommends servicing the product at least once a year.

Service addresses

In the UK
 KG PRODUCTS
 243-251 City Road, Fenton
 Stoke-on-Trent, Staffs
 ST4 2PX
 United Kingdom,
 Phone (44) - 01782 844 866

DIAMOND EDGE
 126 Gloucester Rd.
 Brighton, BN1 4BU.
 United Kingdom
 Phone (44) - 01273 605 922

Or in the US
 A & J Sales and Service
 42 - A Frame Drive
 West Burke, VT 05871
 USA
 Phone: (800) 447-1957

Or in Germany
 Albert Kerbl GmbH
 Felizenzell 9
 84428 Buchbach / Germany
 Phone: +49 (0) 8086 933-100
 Fax: +49 (0) 8086 933-500
 E-Mail: info@kerbl.com
www.kerbl.com

Other service addresses can be obtained from the address indicated above.

7. Troubleshooting list

7.1 Battery-powered clipping machine Favorita_{CL}/Libra

Malfunction	Finding	Cause	Remedy
Battery-powered clipping machine Favorita _{CL} /Libra not starting up	Check switch position	Li-ion battery not charged	Charge the Li-ion battery
	–	Li-ion battery defective	Insert new Li-ion battery
	–	No Li-ion battery inserted	Insert new Li-ion battery
Battery-powered clipping machine Favorita _{CL} /Libra stops running	–	Li-ion battery deactivated by protective circuit	Switch the machine off and on again
	–	Li-ion battery fully discharged	Insert new Li-ion battery
Battery-powered clipping machine Favorita _{CL} /Libra not charged in the charging station	LED indicators remain dark	–	see Chapter Charger and Li-ion battery

7.2 Charger and Li-ion battery

Malfunction	Finding	Cause	Remedy
Charger not functioning	LED indicators remain dark	Power cord not plugged in	Plug in mains power cord in device socket at the charger and in a mains power socket
		Li-ion battery not charged	Replace mains power cord
			Charger defective

Battery-powered clipping machine Favorita_{CL}/Libra

Malfunction	Finding	Cause	Remedy
Li-ion battery not charged	Li-ion battery inserted properly, charge state indicator remains dark	Charger contacts dirty or damaged	Clean the contacts in the charging bay, see Chapter Cleaning/ Disinfecting, or have the charger repaired by the manufacturer
		Li-ion battery defective	Have the Li-ion battery repaired by the manufacturer
		Charger defective	Have the charger repaired by the manufacturer
		Excessive temperature in the Li-ion battery detected during charging	Remove the Li-ion battery from the charging bay, let it cool down and restart charging. If the malfunction recurs, have battery repaired by the manufacturer
		Charging malfunction or Li-ion battery defective	Remove the Li-ion battery from the charging bay and repeat charging, if necessary in another charging bay. If the malfunction recurs, have the Li-ion battery repaired by the manufacturer

8. Technical Service

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

In the UK

KG PRODUCTS

243-251 City Road, Fenton

Stoke-on-Trent, Staffs

ST4 2PX

United Kingdom,

Phone (44) - 01782 844 866

DIAMOND EDGE

126 Gloucester Rd.

Brighton, BN1 4BU.

United Kingdom

Phone (44) - 01273 605 922

Or in the US

A & J Sales and Service

42 - A Frame Drive

West Burke, VT 05871

USA

Phone: (800) 447-1957

Or in Germany

Albert Kerbl GmbH

Felizenzell 9

84428 Buchbach / Germany

Phone: +49 (0) 8086 933-100

Fax: +49 (0) 8086 933-500

E-Mail: info@kerbl.com

www.kerbl.com

Other service addresses can be obtained from the address indicated above.

Battery-powered clipping machine Favorita_{CL}/Libra
9. Accessories/Spare parts

Art. no.	Version	Power cord	Li-ion battery	Charging station	Battery-powered clipping machine	Accessories
GT206	Europe except UK	TA012170 Europe flat connector, black, L=1.8 m	GT201	GT200880	GT200 unpacked	GT604800 Hand oiler
GT206G	UK	TA012169 UK plug, black, L=1.8 m	GT201	GT200880	GT200 unpacked	GT604800 Hand oiler
GT206J	Japan, Mexico 110–125 V	TA012168 2-pole device plug, black, L=1.8 m	GT201	GT200880	GT200 unpacked	GT604800 Hand oiler
GT206K	USA	TA012168 2-pole device plug, black, L=1.8 m	GT201	GT200880	GT200 unpacked	GT604800 Hand oiler
GT206AR	Argentina	TA012333	GT201	GT200880	GT200 unpacked	GT604800 Hand oiler
GT216	Europe except UK	TA012170 Europe flat connector, black, L=1.8 m	GT201	GT200880	GT210 unpacked	GT604800 Hand oiler

10. Technical specifications

10.1 Battery-powered clipping machine Favorita_{CL}/Libra

Model	GT216
Stroke rate	2 300 1/min
Charging/Output voltage	max. 8.4 V
Charging/Output current	max. 1.2 A
Weight (without blade)	400 g
Classification acc. to Directive 93/42/EEC	II
EN 60335-1, EN 60335-2-29	

10.2 Charger

Model	GT203
Mains voltage ranges (Current intake)	100-240 V
AC frequency	50-60 Hz
Charging/Output voltage	max. 8.4 V
Charging/Output current	max. 1.2 A
Weight	495 g
Classification acc. to Directive 93/42/EEC	II
Approval marks	CE, UL

10.3 Li-ion battery

Model	GT201
Cell type	Li
DC voltage	7.4 V
Charge capacity	approx. 1.2 Ah
Charging time	50 min ±5 min

10.4 Declaration of conformity

	We declare under our sole responsibility that this product conforms to the following standards or other normative documents: 2004/108/EC EMC guidelines: DIN EN 60335-1 DIN EN 60335-2-8 Safety of household and similar electrical appliances - general requirements - for hair clipping machines Aesculap Suhl GmbH
---	--

Battery-powered clipping machine Favorita_{CL}/Libra

11. Disposal

Power tools, accessories and packaging should be recycled through environmentally compatible processes.

11.1 For EU countries only

	<p>Do not dispose of power tools with household waste!</p> <p>According to European Directive 2012/19/EU on waste electric and electronic devices, and its implementation in national legislation, power tools that have reached the end of their service life must be collected separately and recycled through environment-friendly processes.</p>
---	--

11.2 Li-ion batteries

Do not dispose of batteries with household refuse and do not throw them into fire or water. Defective or exhausted batteries must be collected and recycled or disposed of in environmentally compatible ways according to Directive 2006/66/EC.

12. 2 Year Warranty

Dear Customer,

Thank you for choosing a product from our company. For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices, using high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Parts that are typically subject to wear, and for which we are therefore not liable, are the battery and clipper head. Defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

13. Distributor in the US/Contact in Canada for product infor- mation and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

Service addresses

A & J Sales and Service
42 - A Frame Drive
West Burke, VT 05871
USA

Phone: (800) 447-1957

Or in Germany

Albert Kerbl GmbH
Felizenzell 9

84428 Buchbach / Germany

Phone: +49 (0) 8086 933-100

Fax: +49 (0) 8086 933-500

E-Mail: info@kerbl.com

www.kerbl.com

Other service addresses can be obtained from the
address indicated above.

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